Diamond Coated Tip Sterilization Instructions

Operation Manual

Rx Only OM-Z0283E 000

Thank you for purchasing this product.

Before use, please be sure to carefully read these Sterilization Instructions and keep it accessible for future reference.

1. User and Indications for Use

User: Qualified Professionals

Indications for Use: This tip is specially designed for use with VarioSurg.

Do not use with other systems. Refer to the Operation Manual for the VarioSurg for specific indications for use.

2. Precautions for Handling and Operation

⚠ CAUTION

- This product must be used by qualified personnel in accordance with the operation manual for the VarioSurg device being used.
- The tip is delivered in a non-sterile condition and must be autoclaved prior to use. Refer to "3. Sterilization before first use" for sterilization conditions.
- **The diamond coated tips are "Single use" items. Do not reuse.**
- U.S. Federal law restricts this device to sale by or on the order of a licensed physician.

3. Sterilization before first use

NSK VarioSurg tips are not sold sterile and must be sterilized prior to first use.

Disregard any sterilization instructions specific to diamond coated tips contained in operations manuals. The following Diamond Coated Tip Sterilization Instructions supersede any instructions regarding diamond coated tips provided in other labeling.

Autoclave sterilization is recommended.

Note the following when sterilizing a diamond coated tip:

- Follow your local directives, standards, and guidelines.
- Sterilize a tip in a U.S. FDA-cleared sterilization pouch, compliant to ISO 11607-1.
- NSK recommends sterilization according to ANSI/AAMI ST79 and EN 13060 (class B).
- Follow the manufacturer's instructions for use that came with your sterilizer.
- For more information about sterilization standards, refer to the medical device regulations specific to your location.

- * U.S.: The recommended sterilization procedures require the use of FDA-cleared sterilizers, sterilization trays, sterilization wraps, biological indicators, chemical indicators, and other sterilization accessories labeled for the sterilization cycle recommended.

■ Autoclave Procedure

- 1) Remove diamond coated tip from the tip case.
- 2) Insert diamond coated tip into a U.S. FDA-cleared sterilization pouch, compliant to ISO 11607-1. Seal the pouch.
- 3) Autoclave conditions are below:

Type	Steam Sterilization Cycle				
	Gravity Displacement		Pre-Vacuum (Dynamic Air Removal)		
Minimum Temperature	132°C	135°C	132°C	134°C*	135°C
Full Cycle Time	15 minutes	10 minutes	4 minutes	3 minutes	3 minutes
Minimum Drying Time	15 minutes	30 minutes	20 minutes	30 minutes	16 minutes

* Not for use in the U.S.

⚠ CAUTION

- Do not exceed 138°C during sterilization.

- 4) Keep the tips in the autoclave pouch until ready to use.
- 5) Store in a dry and dust-free location up to the period specified by the sterilization pouch manufacturer. If sterility cannot be confirmed, sterilize again prior to use.

4. Handling method after use

THE DIAMOND COATED TIPS ARE "SINGLE USE". DO NOT REUSE.

Dispose of diamond coated tips in accordance with local laws and regulations.

5. Symbols

 Caution: U.S. Federal law restricts this device to sale by or on the order of a licensed physician.

 Consult Instructions for Use

 Caution, consult accompanying documents

 Single use. Do not reuse.

Manufacturer

NAKANISHI INC.  www.nsk-inc.com

700 Shimohinata, Kanuma, Tochigi 322-8666, Japan

NSK America Corp www.nskdental.com

1800 Global Parkway, Hoffman Estates, IL 60192, USA

Specifications are subject to change without notice.

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